

SHERASCANSPRAY

INSTRUCTIONS FOR USE

1. Indication

Scanspray for extraoral CAD/CAM use. Reduces the surface reflection thus achieving an ideal scan of the models.

2. Scope of delivery

- 200 ml spray can,
- 1 spray head with angled nozzle for straight to a point application,
- 1 spray head with straight nozzle for application on surfaces.

3. Application

- The surface to be sprayed should be clean and dry.
- Put the spray head on.
- Shake can well before using.
- Hold can vertical while spraying.
- Press spray head always well, thus avoiding plugging of the nozzle.
- Apply a thin spray layer on the corresponding surface from a distance of approx. 5 to 8 cm.
- A thin and complete wetting is achieved by spraying as short as possible.
- After drying of the surface the scan can get started.
- After recording the sprayed surface can be cleaned with water or steam.

4. Security advice

Container is under pressure. Protect against sunlight and temperature of more than 50°C. Even after use do not open by force nor burn. Do not spray into open flames or smouldering objects. Take away from sources of ignition – do not smoke. Keep out of reach of children.

5. Warranty

SHERA Werkstoff-Technologie GmbH is EN ISO 13485 certified and guarantees excellent product quality using a comprehensive quality assurance process. Our user recommendations are based on the benchmark values obtained at our test laboratory. These values can only be guaranteed when adhering to the listed processing phases. The user assumes the responsibility for the processing of the product. We assume no liability for incorrect results as SHERA has no influence over how the product is processed after manufacture. Possible compensation claims are limited solely to the value of our product.

